

Part 1: Charging instructions

- 1.Connect the Micro-USB line to the charging case to let the charging case fully charged.
- 2.When you need to charge the hearing aid,put the machines into the charging case.
- 3.During charge period the indication light will show BLUE.
- 4.Once charging is complete the indication light will show WHITE.
- 5.Charging time: 90min-2 hours
- 6.Operating time: 12 hours



Part 2: Operation Instructions

- 1.Before use,make sure the amplifier is fully charged.
- 2.Taking out the amplifier from the charging case.
- 3.Press up to 3 seconds to turn on when you hear twice “beep”.
- 4.While covering the microphone with your fingertip to avoid the high-pitched sound,place the amplifier into ear.
- 5.Make sure you achieve a comfortable fit,and that the microphone is not being blocked.



- 6.To turn off the amplifier,press up to 3 seconds when you hear twice “beep”.

Volume control:

Press the volume button to adjust the volume. There are 6 sound levels of volume,and you will hear a “beep” sound each time a level goes up.You will hear three times “beep” at the max volume.If you press again,it will go back to the lowest volume.

Part 3: Specification

Maximum OSPL90:	≤120dB+3dB
High frequency average OSPL90:	103dB±4dB
Average gain:	23dB±5dB
Equivalent input noise level:	≤35dB
Total harmonic distortion:	≤10%
Frequency response range:	300Hz~3500Hz
Rated power supply current consumption:	≤6mA
Work time:	12-15hours
Charging time:	90min-2hours

Part 3: Specification

Charging case working current:	≤70mA
Charging case working voltage:	3.7V
Charging case charging time:	90min
Hearing aid charging current:	≤30mA
Hearing aid charging voltage:	4.2V
Hearing aid charging times (double machine):	3-4times

Instruction manual Model: JH-A39

